

Service Solution (SS-0311)

Product: SSS 301-Slide lock Style Chair.

Object: Replace the plastic mechanism shroud and holding pins

Tools: Pliers

Parts: Plastic Mechanism shroud and holding pins (christmans trees) supplied by RGP

Time: Procedure time approximately 10 minutes

Procedures:

1. Remove and discard the existing pins that may be broken off in the mounting holes by grasping with pliers and removing.
2. Mount the mechanism shroud in place carefully aligning the holes in the shroud with the holes in the mechanism.
3. Install the two holding pins (christmas trees) through the shroud holes into the mechanism holes by pushing firmly in with your thumbs, on the pins.
4. Discard broken pins and mechanism shroud- Chair is now ready for operation.

Revised 03/30/5